TREOVIS®

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LUBRICATING OPHTHALMIC SOLUTION

What are TREOVIS?

The unique combination of three active substances contained in TREOVIS : sodium hyaluronate, trehalose and D-panthenol effectively moisturizes and supports the regeneration of the protective barrier of the eye surface. After instillation, the product provides a feeling of long-term comfort.

What are TREOVIS used for?

TREOVIS are intended for intensive moisturizing and supporting the regeneration of the protective barrier of the eye surface when it is dry. They alleviate the symptoms of moderate and severe dry eye syndrome. They accelerate the regeneration of the surface structures of the eye after surgery.

When should I use TREOVIS?

TREOVIS should be used in conditions of discomfort, accompanied by symptoms such as: eye dryness, burning

or tired eyes, which may be caused by:

- dry air in heated or air-conditioned rooms, work at the monitor, long-term TV watching or cigarette smoke;
- mechanical irritation of the eye surface, eg from wearing contact lenses;
- surgical procedures [including laser ones] that interfere with the processes of secretion and proper distribution of the tear film on the surface of the eye.

How do the active ingredients of TREOVIS work? Sodium hyaluronate

A naturally occurring polysaccharide in the human body, including the vitreous body of the eye, is commonly known as a highly moisturizing substance. The mucoadhesive properties of hyaluronic acid ensure better adhesion of the preparation to the eye surface, which contributes to faster healing of the corneal epithelium, while thanks to the viscoelastic (viscoelastic) properties, it increases the stability of the tear film, which is washed out much slower during blinking.

Trehalose

Is a non-reducing disaccharide composed of glucose units, naturally occurring in many living organisms such as plants, insects, fungi and bacteria. It has been identified as a key component of survival under stress related to anhydrobiosis [almost complete dehydration]. Due to its strong protective properties, trehalose has been used in ophthalmological preparations as a substance preventing the stress associated with dehydration of the eye surface, which is a consequence of, among others, dry eye syndrome.

D-panthénol

Is an alcohol analog of pantothenic acid, also called panthothenate or vitamin B5, which is a water-soluble vitamin. Due to its hygroscopic properties and the ability to retain moisture, the topical application of D-panthénol may improve hydration, the effect of which is related to the ability to regenerate the protective barrier.

Who are TREOVIS eye intended for?

TREOVIS are dedicated for adults.

How much to use TREOVIS ?

If necessary, apply one drop into the conjunctival sac. TREOVIS can be used daily, also by users of all types of contact lenses because these drops do not contain preservatives. It is usually applied 3 - 4 times a day, one drop into the conjunctival sac of each eye. You can apply more often if you need.

How to use TREOVIS?

TREOVIS should be used only externally to the conjunctival sac. Before using the preparation, note the date of first opening of the bottle label.

Ne pas utiliser si le conditionnement est endommagé
Do not use if package is damaged
No utilizar si el embalaje está dañado

Attention, se reporter à la notice
Caution, refer to the leaflet
Atención, consulte las instrucciones de uso

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1. Prior to administering the preparation wash hands thoroughly.
2. Remove the cap and apply into the eye. Do not touch the surface of the eye with dropper tip.
3. Tilt your head back slightly.
4. Hold the bottle face down (vertically).
5. Pull your lower eyelid down slightly, pressing the bottle, squeeze one drop into the conjunctival sac.
6. Close your eyelids and spread a drop across the eye surface by moving the eyeball.
7. Repeat the same activities for the other eye.
8. Put the protective cap on the dropper after use. During applying, be careful not to touch the surface of the eye and eyelid with the tip of the dropper.
9. After applying the drops, do not rinse the surface of the eye with water or any other liquid.
10. Remember that a few seconds may pass between the squeezing of the bottle and the flow of the drop in. Do not apply too much pressure.
11. Before closing, point the bottle with the applicator downwards and shake vigorously, without touching the tip of the dropper, in order to remove the remaining drops. Wait a moment before putting the cap on for the bottle to vent.

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CE 0051

When TREOVIS are contraindicated?

- Do not use in the case of hypersensitivity to any of its components.

- Do not use if you are allergic to the ingredients.

What do TREOVIS contain?

Sodium hyaluronate 0.15%, trehalose 3%, D-pantenol 2%, sodium chloride, highly purified water.

How to store TREOVIS?

- Keep out of sight and reach of children.
- Keep away from sources of heat.
- Store in a dry place at a temperature of 5°C to 35°C.
- Do not use the product after the expiration date printed on the package.
- The shelf life of the product is 3 months after first opening the container.
- The product should not be used for more than 1 month in continuous (the prescription can be renewed if needed), and not more than 3 months in discontinuous use.
- Discard any unused solution.

Warnings and precautions:

- The preparation is intended for external use only.
- Do not use if package is damaged.
- Never touch your eye or any other surface with the tip of the dropper, this may contaminate the solution.
- In the case of using other ophthalmic solutions should wait at least 15 minutes before the application of TREOVIS.
- The preparation should be administered last.
- The reusable medical device should be used by one patient.

Reporting medical incidents or adverse reactions:

If you get any side effects, please tell your doctor, pharmacist or nurse.

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